VITA-diffuser™

GB USER INSTRUCTIONS for
VITA-diffuser™

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INSTRUCTIONS FOR USE FOR VITA-DIFFUSER™

REF 3014

VITA-diffuser™ is always to be used in conjunction with either the ST310 or ST320 surgical humidification kits from Fisher & Paykel Healthcare – see separate instructions for use enclosed.

Intended use:

- 1) For the distribution of humidified and heated CO₂ during open surgery in order to avoid dehydration and cooling of the operative wound.
- 2) To be used by cardiovascular surgeons in open heart surgery procedures for insufflation of CO₂ into the thoracic cavity to reduce the risk of air embolism. These instructions are intended for general use of VITA-diffuser™. Variations in use may occur in specific procedures due to individual technique and patient conditions.

Before opening, please check the package for external damage. In case of damage, please discard the package.

Preparations:

- 1) Ensure that a CO₂ source is available with the capacity to deliver 10 litres/minute via a flow meter for the estimated duration of the operation.
- 2) The flow meter must be connected to or contain a pressure regulator. The working pressure for the CO₂ gas must be between 3.5 and 4.5 bar (50-65 psi or 350-450 kpa).
- 3) Read the enclosed instructions for use for connecting and starting up the ST310/ST320 system.
- 4) Connect the VITA-diffuser™ hose with the ST310/ST320 from Fisher & Paykel Healthcare.
- 5) Place the foam body part in the operative wound. Bend the narrow hose so that the foam body part is located about 5 cm below the edge of the incision.
- 6) Then secure the hose.
- 7) If the operative wound is <5 cm deep, the operation cavity should be artificially deepened in order to gain maximum effect. This is done by creating a high enough wall around the incision edge.

Starting and supplying the flow:

- 1) Start a CO₂ flow of 10 litres/minute as soon as the VITA-diffuser™ has been placed in the operative wound and connected with the ST310/ST320 as per the separate instructions for use.
- 2) Maintain a supply of 10 litres CO₂/minute throughout the procedure until the incision is closed. Then switch off the flow.
- 3) In order to prevent air embolism during open-heart surgery, it is necessary to supply a flow of CO₂ at 10 litres/minute from 1 minute before the heart or the major blood vessels are opened and continually until the heart and the major blood vessels have been closed.
- 4) If the foam body of the VITA-diffuser™ becomes completely soaked in fluid, briefly squeezing the foam body will restore its function.
- 5) Avoid using a suction rate that exceeds the gas flow rate of CO₂. Otherwise there is a risk that room air may enter the operative wound.
- 6) Do not exert excessive traction force on the device.

After use:

- 1) Dispose of VITA-diffuser™ as blood-contaminated material.
- 2) Ensure that the device has not been severed by any cut or excessive traction force.
- 3) VITA-diffuser™ is intended for single use only and should not be cleaned and reused.
- 4) For ST310/ST320 surgical humidification kits, see separate instructions for use.

Storage:

In a location that is free from damp, away from direct heat and sunlight.

COMPONENT	MATERIAL
Hose	PVC
Flexible hose section	PVC and stainless steel
Foam	Polyurethane
Adhesive	Cyclohexanone
TECHNICAL DATA	
Gas flow rate	10 litres/minute is recommended
Storage temperature	10-30°C, 50-86°F
Sterilisation method	Ethylene oxide

Warranty and limitation of liability:

- A) The manufacturer guarantees that VITA-diffuser™ has been manufactured in accordance with its stated specifications and in compliance with good manufacturing practice, other applicable industrial standards and regulatory requirements.
- B) The manufacturer will replace, free of charge, a VITA-diffuser™ that has a manufacturing defect, provided that this is detected and reported within twenty-four (24) months of delivery and that the product batch number can be stated.
- C) The manufacturer shall not be held responsible for incorrect use, improper handling, ignorance of warnings, precautions and instructions, damage resulting from events occurring after the manufacturer has delivered VITA-diffuser™, failure to inspect or defective inspection of VITA-diffuser™ before or after use in order to ensure that the VITA-diffuser™ is in proper condition, or any warranty that is given by an independent distributor or dealer.

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E-mail: contact@cardiainnovation.com, website: www.cardiainnovation.com. © 2013

The product is protected by the following patents, held by Cardia Innovation AB Patent#US6494858B1, 6994685B2, 7549973B2, Patent#Europe 1032322, 1239915, patent#Sweden515473

Explanations of symbols on the product label.

Manufacturer

Batch number

Temperature limits

TO BE DISCARDED IF
PACKAGING IS DAMAGED

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CARDIA INNOVATION AB.